


Digital Dentures From Physical Impressions

Conventionally made dentures have historically been a pain-point for many dental practices. Lengthy turnaround times and adjustments go hand-in-hand with dentures made the old fashioned way. Digitally printed dentures will change all of that! Whether you take impressions **conventionally or digitally**, Great Impressions can make you a premium, 3d printed high-impact denture. Below is the process to make that happen.

Appointment 1: Preliminary Impression

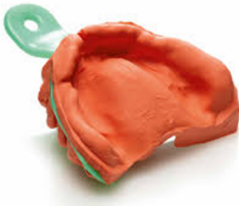

Clinician: Take a PVS of the patient, ensuring that as much ridge as possible is captured.

Lab: Fabricates occlusal rim(s).

Appointment 2: Occlusal Rim Records



Clinician: Place occlusal rim(s) in patient's mouth and make necessary markings to establish VDO, mid line, centric relations, & lip line at rest (horizontal markings).

Lab: Fabricates monoblock try-in based on input from occlusal rim(s).

Appointment 3: Monoblock Try-in



Clinician: Place try-in(s) in patient's mouth and ensure correct fit, teeth positioning, bite, etc... If adjustments are needed, grind on monoblock or add composite for us to copy, or you can notate and take pictures. Take a new bite if occlusal adjustments are needed.

Lab: If try-in is approved, the final denture will be fabricated.
If adjustments are needed, an additional try-in may be necessary.

Appointment 4: Final Delivery



Clinician: Deliver Completed Denture!

Lab: All designs are saved on GIDL servers. Dentures can be replicated at the click of a button if the patient loses or damages the appliance.

But wait there's more! We also have the ability to copy a patient's current dentures. Call us to learn more!